
Bosworth Softone Instructions for Use

Softone is a tissue conditioner and functional impression material that conditions, massages, and rehabilitates damaged tissue before a new denture rebase or final relines.

DIRECTIONS

1. Clean and dry denture thoroughly. Roughen denture surface and periphery for best adhesion.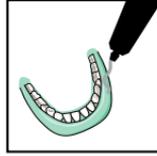

2. Coat buccal, labial and other surfaces NOT to be bonded with BOSWORTH MASQUE.

3. Add 1 full vial of liquid to 1 full plastic vial of powder in mixing cup. Spatulate for 1 minute. Mix slowly to reduce bubbles. Let stand for 15 seconds.


4. Pour into denture, spreading evenly with spatula over entire surface. Wait 2-3 minutes.

5. Place denture in patient's mouth and have patient gently close to align occlusion*. Muscle trim. Let sit for 4-5 minutes.

*Caution: Apply pressure to labial surface to guard against protrusive positioning.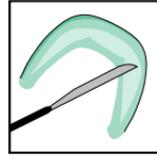

6. Remove and trim excess with heated spatula, sharp knife or scissors. For final polish use very wet slurry of pumice at low speed.

*Caution: Do not heat up denture by holding against buff wheel for long periods.

NOTES

1. Softone relines may be cleaned by rinsing with a mild, alcohol-free cleaner or by brushing with a soft toothbrush and a mild non-abrasive toothpaste.
2. Polishing may be expedited by first chilling denture in refrigerator or ice water.
3. Softone is a temporary impression material formulated to last up to 4 days in the mouth.

CAUTION: Certain individuals may be sensitive to products containing acrylic.

MIX RATIO

6,8ml (1 full glass vial) liquid to
9,0gm (1 full plastic vial) powder

CONSISTENCY

46.1 millimeters

PENETRATION RATIO

3,61

The penetration ratio indicates the change in firmness of the material between 2 hours and 7 days. Penetration ratios greater than 1 indicate increased firmness. Ratios less than 1 indicate that the material is softer at 7 days.

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