Bosworth Acid Etch

37% Acid Etchant

37% Phosphoric Acid / 37% Gel d'acide phosphorique / 37% Phosphorsäure-Ätzgel
37% Gel di acido fosforico / 37% Gel de ácido fosfórico /
37% Gel de ácido fosfórico / 37% Fosforsyregel

Danger

Causes severe skin burns and eye damage. Refer to product SDS for additional health and safety information.

CAUTION: Federal law restricts this device to sale by, or on the order of a dental professional.

Keystone Industries
52 West King St.
Myerstown, PA 17067 USA
800.333.3131 keystoneindustries.com


keystoneindustries.com/symbolglossary

Made in USA
BOSLAB-207 PR805 12/2019

ORDERING INFORMATION
INFORMATIONS CONCERNANT LES COMMANDES
BESTELLINFORMATIONEN
INFORMAZIONI PER LE ORDINAZIONI
INFORMAÇÃO PARA ENCOMENDA
BESTÄLLNINGSSINFORMATION
INFORMACIÓN DE PEDIDOS

[REF] 0921346 Standard Kit

Kit de coupe standard • Standardsatz • Kit standard
Kit padrão • Kit estándar • Standard släppt upp

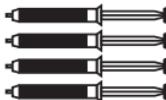Box of 4 - 3ml syringes Acid Etch 37% Phosphoric Acid Syringes

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Packung mit vier Spritzen à 3 mL

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Caixa de quatro seringas de 3 mL

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10 - Tips

10 - Embouts

10 - Applikatortips

10 - Puntali

10 - Pontas

10 - Puntas

10 - Spetsar

10 - Placement Brushes

10 - Des Pinceaux

10 - Bürsten


10 - Spazzole

10 - Pincéis


10 - Pinceles

10 - Borstar


Bosworth Acid Etch Instructions for Use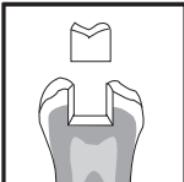


1. Prepare the tooth to the desired configuration.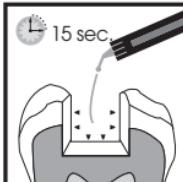


2. Attach Flowthru® Microbrush®-style tip to the etchant syringe and dispense etchant gel either directly onto the tooth surface or onto pad and brush onto tooth surface. The etchant should be placed first on enamel surfaces, then on the dentin.

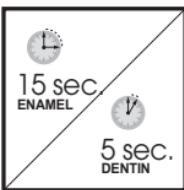
3. Recommended etching time for enamel is 10-15 seconds. An adequate etch of the dentin can be achieved in 5 seconds.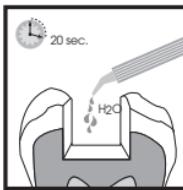

4. Rinse well with water for 10-20 seconds removing all of the etchant material.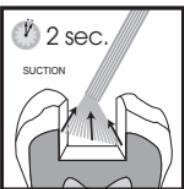

5. Remove all excess moisture by placing high volume suction over the prepared surfaces for 2 seconds. The prepared tooth surface should be visibly moist, but not desiccated or saturated with water.

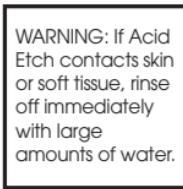CAUTION

1. Acid Etch contains phosphoric acid which may cause burns or skin sensitization in susceptible individuals.
2. Before using this product cover the patient's eyes with protective eyewear.
3. Avoid contact with oral tissues, eyes and skin. If Acid Etch contacts eyes, skin or soft tissue, rinse off immediately with large amounts of water while using an oral evacuator or saliva ejector and seek medical attention.
4. Acid Etch should extrude easily from the syringe. DO NOT USE EXCESSIVE FORCE. Excessive pressure may result in unanticipated extrusion of the material.
5. Do not use Acid Etch on dental pulp tissue; cover dentin close to the pulp (< 1 mm remaining) with a hard setting calcium hydroxide liner.

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