

MiiCraft 125 Printer Instructions

KeyGuide™ / KeySplint™ Hard Resin

Print Settings	
Thickness (µm)	50
Curing Time (Model Layers)	4.9
Speed	Advanced / File
Gap Adj (mm)	0
Base Layers	1
Base Curing(s)	20
Buffer Layers	8

Advanced/ File Setting	
Step 1	Cartridge / Down / 1000 / 800
Step 2	Picker / Up / 1400 / 800
Step 3	Picker / 1 Layer / 0 / 800
Step 4	Cartridge / Up / 1000 / 800
Step 5	Picker / Down / 1400 / 1600
Step 6	