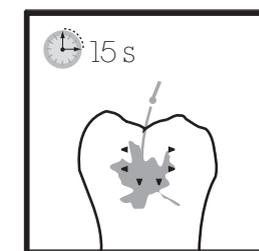
# Bosworth Aegis® Pit & Fissure Sealant

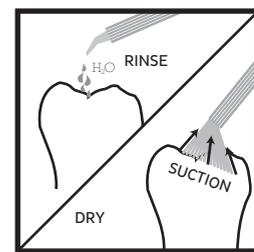
## Light-Cure Sealant with ACP

AEGIS PIT & FISSURE SEALANT is an opaque 38% filled light-cured resin with Amorphous Calcium Phosphate (ACP). The sealant cures to an opaque film on enamel. ACP will provide a source of calcium and phosphate ions.


**BEFORE USING PIT & FISSURE SEALANT:** Clean teeth to be sealed thoroughly using your favorite method. (Air abrasion or a thin diamond bur may be used to clean fissures.) If necessary, use a rubber dam to isolate teeth.


1. Apply ACID ETCH for 15 to 20 seconds on enamel.

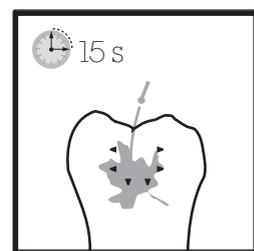
2. Rinse ACID ETCH off with water. Dry with suction or clean oil-free air.

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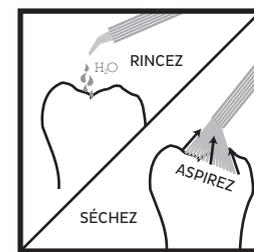
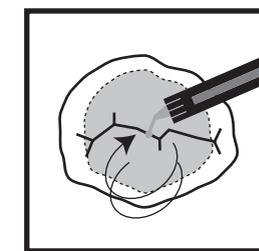
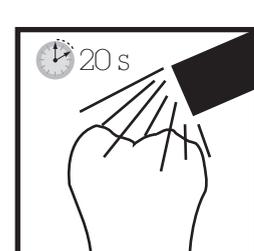
3. Apply PIT & FISSURE SEALANT directly to tooth with the disposable application tip or use a disposable brush.

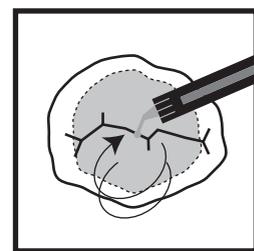4. Cure PIT & FISSURE SEALANT with light for 20 seconds. Final Set time is 30-40 seconds.

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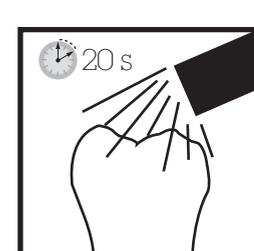### NOTE

After each use, dispose of dispensing tip and recap syringe. Prior to next use, attach a new disposable tip.

**CURING TIMES:** Use a standard high-intensity curing light (intensity greater than 300 mWatt/cm<sup>2</sup> in the wavelength range of 300-500 nm). Lower powered lights can require longer curing times while very high intensity lights can cure in shorter times.

### CAUTIONS

1. Do not use PIT & FISSURE SEALANT on persons with a sensitivity to acrylics.
2. Wear protective gloves and eye protection when using this product.
3. See SDS for safe handling precautions and proper waste disposal.

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