Gelato Prophylaxis Paste

Contains: Gelato Dental Prophylaxis Paste with fluoride contains a unique blend of polishing and cleaning agents. Gelato Prophylaxis Paste is for professional use only and is designed for application during standard dental practice hygiene procedures. This product contains fluoride at a concentration of 1.23% fluoride ion.

Warnings: Excessive ingestion of fluoride may cause nausea. Gelato Dental Prophylaxis Paste should be expectorated and NOT SWALLOWED. If pregnant or breast feeding, ask a health professional before use. Keep out of reach of children. In case of overdose, get medical help or contact a Poison Control Center right away.

Precautions: Gelato Prophylaxis Pastes are to be used only by individuals professionally trained to perform a dental prophylaxis treatment. During use, wear protective glasses and gloves.

Suggested Applications:

Fine: Gentle polishing and cleansing. Recommended for children.

Medium: Normal polishing, cleansing, and plaque removal.

Coarse: Medium to heavy stain and plaque removal.

X-Coarse: For heavy stains and plaque removal.

Directions for use:

Bulk Jars: Remove sufficient paste for one procedure and place in a clean dappen dish or other suitable container.

Unit Dose: Each unit dose cup contains enough Gelato Prophy Paste for one (1) prophylaxis procedure.

Use any commercially available prophy angle, rubber cup combination. To fill the cup press the cup face into the paste while running the angle at low speed. To avoid overheating the tooth surface, maintain sufficient paste in the cup throughout the procedure. Evaluate saliva to avoid swallowing and/or excessive dilution of paste.

Storage:

Store product in a cool, dry place. Keep out of direct sunlight.

Manufactured by:
Keystone Industries

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