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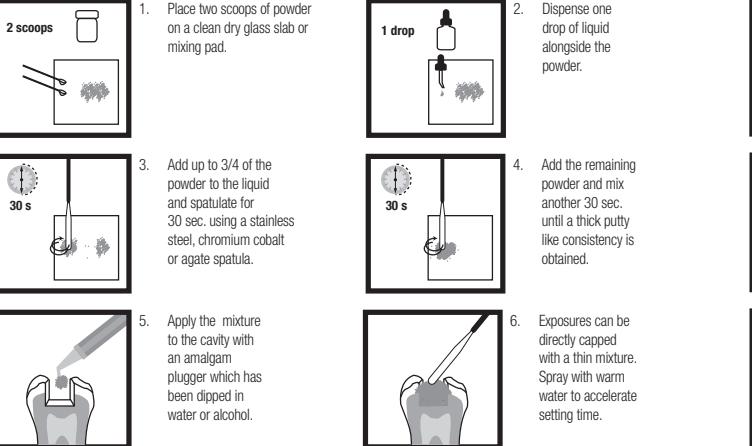
Bosworth SuperEBA® Instructions for Use

EN


Indications for Use: SuperEBA is a two-part (liquid and powder) mixture for use as a temporary or permanent dental cement, or as a base or liner in dental restorations. **Warnings/Precautions:** Product contains zinc, eugenol and rosin, which although rare, may cause an allergic reaction in some patients sensitive to those ingredients. Clinicians should use care in screening their patients for any known allergies. Liquid and powder causes serious eye and skin irritation. The liquid is harmful if swallowed. Use appropriate personal protective equipment when mixing and handling. Refer to SDS for additional information. (Combine all warnings under one section heading)

Disposal: Waste is harmful to aquatic life with long lasting effects and should not enter sewage system. Dispose of unused materials in accordance with federal, state, and local regulations. **Storage:** Keep containers tightly closed and protected from direct sunlight when not in use and store in a dry area at normal room temperature.

DIRECTIONS FOR LININGS

DIRECTIONS FOR FINAL CEMENTATION

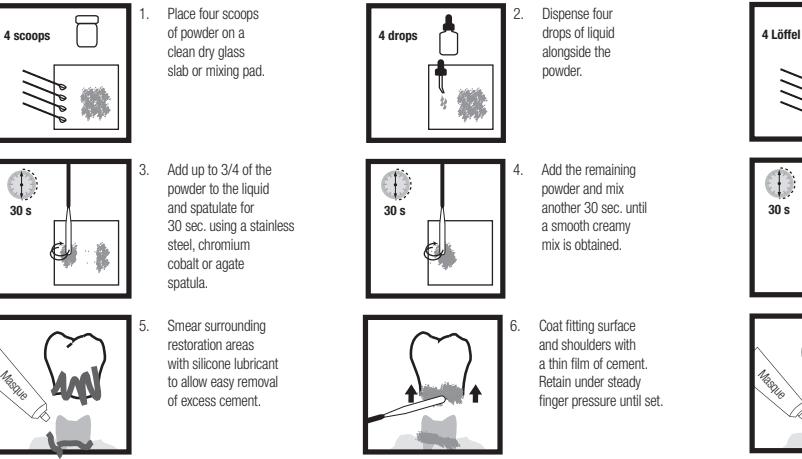SETTING TIME (MOUTH)

- Regular (Slow) Set 10-15 minutes
- Fast Set 4-8 minutes

SETTING TIME (SLAB)

- Regular (Slow) Set >15 minutes
- Fast Set 10-15 minutes

TIPS FOR USING SUPEREBA®

1. The degree of latitude in mixing SuperEBA® is very wide and will be affected by the temperature and humidity in your operatory. Higher temperature and humidity may shorten work and set times. Lower temperatures and humidity may increase work and set times.
2. As you become accustomed to this cement, with its improved flow, you will observe that thicker mixes than those recommended below can be used for some cementing procedures, e.g. Orthodontic cementation.

Liquid	
1) Ortho-Ethoxy	62.5%
Benzoic Acid	
2) Eugenol	37.5%
Powder	
1) Zinc Oxide	60%
2) Alumina	34%
3) Natural Resin	6%

Recommended Powder/Liquid Ratios	
Lining	.26 g powder - .03 g liquid
Cementing	.52 g powder - .12 g liquid

For use by a dental professional only.

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