EN

Itsoclear

HIGH QUALITY CRYSTAL CLEAR CLASP

Itsoclear provides you with the ability to produce and repair a clasp in under 4 minutes with a cosmetic appeal unlike any other clasp.

Itsoclear's superior difference stands out from all the rest

- Saves bench time producing a higher quality dental esthetic clasp that your customers will appreciate while increasing your profit.
- Crystal clear, will not lose its clarity
- Non-abrasive
- Easily adjustable
- No injection machines or hand-held injectors required
- Perfect for new partial dentures as well as repairs and replacing metal clasp

4 easy steps to create your Itsoclear clasp in under 4 minutes

- 1A) Warm clasp by lightly waving it over flame. (WARNING: Do not overheat the clasp)
- 1B) Lightly pass 3 prong plier over flame. (WARNING: Overheating plier will cause indentation in clasp)
- 1C) Adapt accordingly around tooth by rewarmed the 3 prong plier as needed.
- 2A) After clasp is formed around tooth begin to create retention.
- 2B) Form retention by bending a loop at opposite end of clasp to hold in acrylic.
- 2C) Once retention is formed the clasp is ready to be set in place on frame.
- 3) Secure clasp on frame by using glue and accelerator.
- 4) Finally, place acrylic under retentive area and above the class so that the clasp will remain in position. Your partial denture is now ready to be processed as normal.

REPAIRS or REPLACEMENTS: Cut groove where clasp is to be formed and cut a deep groove in tentative area. Process as normal.

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EC REP Keystone Europe LLC