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
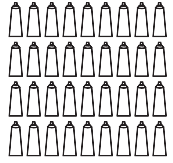
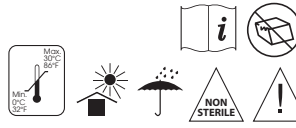
0214PR112


## Bosworth Cora-Caine Instructions For Use

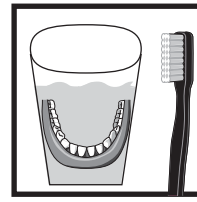Cora-Caine provides a secure fit by soothing tender ridges and relieving the discomfort of sore spots. Acting as a cushion between the denture and the tissue, Cora-Caine provides positive retention and enhanced stability. Cora-Caine is pink in color, has no smell or aftertaste, and remains effective up to 24 hours.

### PRIOR TO USE

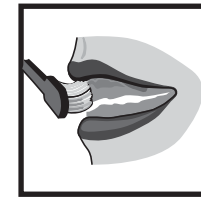
1. Clean and dry denture.


2. Thoroughly clean and rinse the patient's mouth.

### DIRECTIONS FOR USE ON UPPER DENTURES

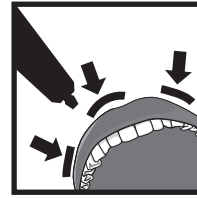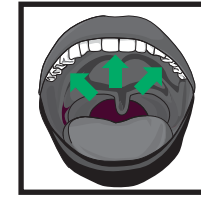
1. Flow a 1/4 inch (6mm) ribbon of Cora-Caine into the right, left and center ridge area of upper denture.


2. Replace the denture slowly and wipe away excess.

### DIRECTIONS FOR USE ON LOWER DENTURES

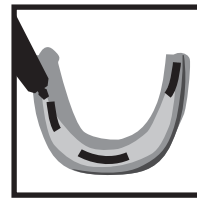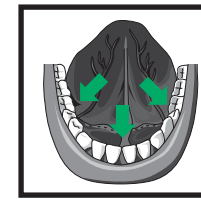1. Flow a 1/4 inch (6mm) ribbon of Cora-Caine into the right, left and center ridge area of lower denture.

2. Replace the denture slowly and wipe away excess.

### CAUTIONS

- Prolonged use of this medication is not recommended as adjustment of the denture by your dentist is necessary to maintain good oral health. Do not exceed recommended dosage or use for more than 7 days unless directed by a dentist or doctor. Discontinue use and consult a doctor if sore mouth symptoms do not improve or if irritation worsens and swelling, rash, or fever develops.
- Patients with a known history of hypersensitivity to "caine" products should not use Cora-Caine®. Localized allergic reactions may occur after prolonged or repeated use. Do not use this product if you have a history of allergy to local anesthetics such as procaine, butacaine, benzocaine, or other "caine" anesthetics.
- Avoid contact with the eyes.
- Get medical help or contact a Poison Control Center immediately in the case of overdose or ingestion.
- Keep out of reach of children.

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