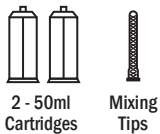


**ORDERING INFORMATION**

0921315 Standard Kit  
0921316 Standard Kit (Rigid)



**Bosworth Company**

Please forward health & safety questions to:  
Veuillez nous faire parvenir vos questions en matière de santé de sécurité à:  
Fragen zur Gesundheit und Sicherheit richten Sie bitte an:  
Inviate domande relative alla sicurezza alla salute a:  
Para quaisquer questões de saúde e segurança contactar:  
For more information, contacta din your dental distributor eller:  
For favor dirija preguntas salud y seguridad a:

The Bosworth Company  
7227 North Hamlin Avenue  
Skokie, IL 60076-3999 USA or  
F. Moore  
Unit 3, Haldene Way  
Seaham Grange Ind. Estate, Seaham  
CO Durham, SR7 OPU England  
www.bosworth.com • hbinfo@bosworth.com



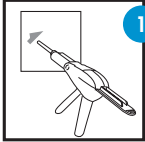
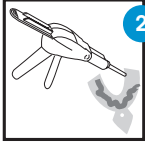

For Dental Use Only

Made in USA  
Fabriqué aux États-Unis  
Hergestellt in den USA  
Fabricato in USA  
Fabricado nos E.U.A.  
Gjord i USA  
Fabricado en EE. UU.  
0514PR140

**Bosworth Re'CORD Instructions For Use**

Re'CORD is a specially formulated bite registration material designed to provide enhanced accuracy, convenience, and patient acceptance.

**INSTRUCTIONS FOR USE**

-  Insert cartridge into dispenser. Extrude a small amount of material to ensure even mix.<sup>1</sup>
-  Attach mixing tip and apply directly into mouth or onto tray.<sup>2</sup> Insert tray in patient's mouth. Leave mixing tip attached until next use.
-  When set remove from patient's mouth. Clean and air dry.<sup>3</sup>

**NOTES**

- Remove any clogs with an explorer and bleed the cartridge until material begins to flow freely.
- Areas where teeth have been prepared should be injected first. Usually, only the teeth of one arch, whether the upper or lower, need to be covered. In situations where a tray is desired, material is placed on both sides of a Bosworth Big Bite Tray or other similar mesh tray.
- The impression may be safely disinfected by immersion in a 2% glutaraldehyde solution.
- Wait ten minutes before pouring stone model. For best results, delay pouring model for at least 12 hours.

**CAUTIONS**

Certain types of latex or vinyl gloves can interfere with the set of vinyl products. It is suggested the operator check the compatibility of their brand of gloves prior to any impression procedure.

Can cause irritation or other allergic reactions in sensitive subjects. Do not use after expiration date. Do not leave residual bite registration material in the sulcus or oral cavity. Do not swallow bite registration material! Avoid contact with the eyes. Avoid contact with clothing. Do not allow material to enter the sewage or water system.

|                 | WORKING TIME<br>23°C (73°F) | INTRAORAL<br>SETTING TIME | FINAL HARDNESS |
|-----------------|-----------------------------|---------------------------|----------------|
| <b>STANDARD</b> | 90 seconds                  | 75 seconds                | 75 Shore A     |
| <b>RIGID</b>    | 30 seconds                  | 45 seconds                | 80 Shore A     |