

# TROUBLE SHOOTING - MOUTHGUARD STRAP

Available in: Black #9597340 / Clear #9597400

1. Both surfaces need to be hot.

Once mouthguard is formed, there is about 1 ½ min. prime time to attach the strap.

2. After attaching strap to the labial surface, use a wet paper towel to push up against the buckle flange to force any air out that may have been trapped.

3. After fusing strap onto mouthguard, use hot knife or ProForm finishing wheel to feather strap edge into mouthguard.

It is BEST to attach the strap while the mouthguard is still hot & on the model.

However, IF the mouthguard you are attaching a helmet strap to a mouthguard that has already been cooled; Please follow instructions below:

1. Place mouthguard back onto patients model. (to ensure keeping original mouthguard form)
2. Heat a hot wax spatula over a Bunsen burner until hot; run along anterior of mouthguard.
3. Be careful not to overheat or distort the anterior.
4. Use a microtorch to heat tab of strap and position onto mouthguard while anterior is hot.
5. Once again heat the spatula and use it to feather the strap into the mouthguard.

