

Soft EVA (Suave EVA)

This material is made of a clear soft ethyl vinyl acetate. It is available in clear or colors, and is used for mouthguards, niteguards, model duplication, bleaching trays, soft brushing appliances, or custom fluoride trays.

INSTRUCTIONS:

- 1) Heat Material. The material will begin to droop below the bottom material plate. Thinner materials should droop approximately $\frac{1}{4}$ " to $\frac{1}{2}$ ". Thicker material should droop approximately 1". When the material has been appropriately heated, lower the material plate onto the model and turn on the vacuum. Allow the material to cool for approximately 10 minutes before removing from model. Improper heating can result in the formation of creases.
- 2) Trim with scissor. For finishing appliance, put back on model and flame edges with micro torch or use rubber point.

NOTE: for mouthguards, trim as high into the vestibule as possible, and cover all occlusal surfaces.

Este material está hecho de un acetato etileno de vinil suave transparente. Está disponible en claro o en colores, y se usa para protectores bucales, guardas nocturnas, duplicación de modelos, cubetas para blanqueamiento, dispositivos para bruxismo suave o cubetas de fluoruro personalizadas.

INSTRUCCIONES:

- 1) Calentar el material. El material comenzará a caer por debajo de la placa de material inferior. Los materiales más delgados deben caer aproximadamente $\frac{1}{4}$ " a $\frac{1}{2}$ ". El material más grueso debe caer aproximadamente 1". Cuando el material se haya calentado adecuadamente, coloque la placa de material sobre el modelo y encienda el vacío. Deje que el material se enfríe durante aproximadamente 10 minutos antes de retirarlo del modelo. El calentamiento incorrecto puede dar lugar a la formación de pliegues.
- 2) Recortar con tijera. Para el acabado del protector, vuelva a colocar sobre el modelo y caliente los bordes usando un microtorch o use punta de goma.

NOTA: para los protectores bucales, recorte lo más alto posible en el vestíbulo, y cubra todas las superficies oclusales.



Store the product in a cool, dry place. Almacene el producto en un lugar fresco y seco.

MADE IN
USA

Fabricado en USA

PROLAB-012 7190240 03/2019