



Mizzy Pressure Indicating Paste

Pressure Indicating Paste (PIP) is a pure white silicone paste, used for seating crowns, bridges and full, immediate and partial dentures.

About PIP

The brief procedure yields specific results that are easy to read and thoroughly reliable, surpassing results obtained with chemical set pressure indicating materials used to locate areas of denture discomfort. PIP is applied to the

dry denture; companion PIP spray is applied to the tissue. The denture is then seated under pressure then removed. Areas where the paste has been displaced show points of excessive pressure.



keystone[®]
industries

dental.keystoneindustries.com

US: 480 South Democrat Road Gibbstown NJ, 08027
EU: Keystone Industries GmbH Stockholzstr. 11 78224 Singen, Germany

US: 1.800.333.3131
EU: +49 77 31 91 21 01