



Super EBA®

SuperEBA® is an all-purpose, alumina-fortified material for the final cementing of a single crown or inlay. It can also be used as a base, liner, or temporary dressing for pain relief.

About Super EBA

SuperEBA's neutral pH and mild formulation provide additional patient comfort during the healing process. Its hydrophilic nature and low film thickness provide unsurpassed convenience, making it an excellent choice for sensitive patients. It adheres to tooth structure in the presence of moisture. Not to be used for implants.



Tips

Regular set has a slab set time of over 15 minutes, and a shorter in-the-mouth set time of 10-15 minutes. The fast set has a 10-15 minute slab set, and a faster in the mouth set of 4-8 minutes. Film thickness 25 microns.

PRODUCT	QUANTITY	ITEM NUMBER
Standard Kit		0921007
Standard Kit (All Fast Set)		0921007F
Standard Kit (All Regular Set)		0921007R
Powder Fast Set	15g	0921008
Powder Regular Set	15g	0921009
Liquid Only	6.5ml	0921010
Powder Fast Set	2 oz	0921012
Powder Regular Set	2 oz	0921013



dental.keystoneindustries.com