

# SEITY

## 37% Phosphoric Acid Etchant Gel

### INSTRUCTIONS FOR USE

Seity Etch is a thixotropic 37% phosphoric acid etchant for etching enamel, dentin and glass ionomer cement. The thick gel stays in place and will not run onto exposed tissue. The Etchant Gel syringes with the pre-bent small gauge applicator tips facilitate controlled direct extrusion for faster procedures and easier access into difficult to reach areas. The inorganic filler particles in the Seity Etch wash away with water without leaving a residue.

Seity Etch is available in dark blue.

### INSTRUCTIONS:

1. Isolate tooth. Prepare cavity. Clean the surface to be etched using a non-fluoridated, oil-free prophylaxis paste, or a pumice and water slurry. Rinse thoroughly with water and dry.
2. Securely attach the luer lock pre-bent applicator tip onto the syringe. Start flow on pad; not in patient's mouth. Do not push on plunger with thumb. For safe, controlled dispensing, rest plunger in palm of hand and pull back on syringe barrel with thumb, index and middle fingers. Do not force plunger down into barrel.
3. Recommended etching time for enamel is 20-30 seconds and for dentin and glass ionomer liner, 20 seconds. To the extent possible, apply Seity Etch first to enamel and then to dentin. Leave on dentin for 20 seconds.
4. Rinse off Seity Etch with copious amounts of water. Do not disturb the etched surface.
5. Leave surface wet, or dry surface with clean, uncontaminated air, depending on the instructions for use for the bonding system you are using.
6. Cap syringe immediately after use.
7. Remove used tip and attach new disposable tip before each new patient use.

**NOTE:** Etched enamel should appear chalky white. If not, reapply etch gel to enamel for 20 seconds and rinse with copious amounts of water.

# SEITY

## 37% Phosphoric Acid Etchant Gel

### PRECAUTIONS:

Avoid contact with oral tissue, eyes and skin. If accidental contact occurs, wash with copious amounts of water. In case of eye contact, wash eye for 15 minutes and seek medical attention. Keep out of reach of children. Do not take internally. When not in use, keep the syringe tightly capped. Use at room temperature. Do not use after expiry date.

**CAUTION:** Avoid contact with skin and eyes. Keep out of reach of children. Federal Law restricts this device to sale by or on the order of a dentist.



#### Seity Etch Kit (Blue)

Reorder #07-02377  
4 x 1.2ml syringes  
20 pre-bent disposable applicator tips

#### Seity Etch Bulk Kit (Blue)

Reorder #07-00277  
25 x 1.2ml syringes

#### Seity Etch Syringe (Blue)

Reorder #07-02777  
1 x 12g syringe  
20 pre-bent disposable applicator tips

#### Seity Etch Bulk Kit (Blue)

Reorder #07-00377  
1 x 50ml syringe  
4 x 1.2ml empty syringes  
50 pre-bent disposable applicator tips

#### Pre-bent Disposable Applicator Tips

Reorder #07-02977  
1 case x 25 pre-bent tips 25ga



Manufactured by  
Keystone Industries  
480 South Democrat Road, Gibbstown, NJ 08027  
1-800-333-3131 • www.keystoneind.com

99115208 Rev 11/2014