


If the nozzle on this can clogs, turn can upside down and spray one second. Rotate nozzle a quarter turn and return to original position. If this procedure fails to unclog the nozzle, simply replace the nozzle with the extra one provided. Never stick a pin, nail or any sharp object into opening on top of can. Clean up with Isopropyl Alcohol.

1. Spray tray liberally.
  2. Wait 3-5 minutes to place impression material into tray.
1. Pulverice la cubeta generosamente.
  2. Espere de 3 a 5 minutos antes de colocar el material de impresión en la cubeta.
1. Vaporiser une généreuse quantité d'adhésif dans le porte-empainte.
  2. Attendre 3 à 5 minutes, puis déposer le matériau d'empreinte dans le porte-empainte.

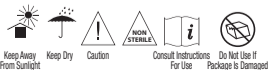

**DANGER:** Extremely flammable gas. Contains gas under pressure; may explode if heated. Propellant: propane and butane.

**CAUTION:** Keep away from heat/sparks/open flames/hot surfaces. No smoking. Use in a well-ventilated area. Do not spray in eyes. Keep out of reach of children.

**REFER TO SDS FOR ADDITIONAL HEALTH AND SAFETY INFORMATION.**

**MADE IN USA. Fabricado en EE.UU. Fabriqué aux É.-U.**