


# Bosworth Big Bite Trays™ Instructions for Use

Big Bite Trays are economical, disposable, plastic frames with a gauze insert to be used when recording centric occlusion. Big Bite Trays can be used on either side to cover a quadrant or for bi-lateral registration by clipping together right and left trays. The thin gauze mesh does not interfere with closure and ensures an accurate bite.

- 1 Mix registration paste or product of your choice.
- 2 Apply a thin layer of mix to one side of the Big Bite Tray with a spatula. Repeat on the other side of the Big Bite Tray.
- 3 Smooth surface on both sides with a spatula.
- 4 Insert tray, making sure that the distal rim of the tray is out of occlusion.
- 5 Have patient close into centric until set.
- 6 When material has set, remove tray.

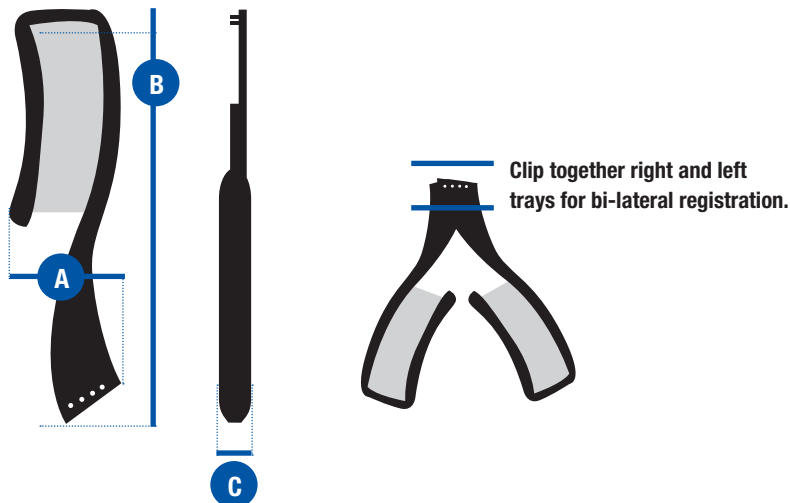

## DIMENSIONS

A width 28.5 mm | B height 44.54 mm | C depth 9.5 mm

# Bosworth Big Bite Trays™

Disposable Bite Registration Trays

## **Bosworth® Company**

### ORDERING INFORMATION

---

0921841    Box of 50 Trays (25 left, 25 right)

## **Bosworth® Company**

Please forward health & safety questions to:

**The Harry J. Bosworth Company**

A division of Keystone Industries  
52 West King St.

Myerstown, PA 17067

800.333.3131

[www.keystoneind.com](http://www.keystoneind.com)

**EC REP**

**Keystone Europe LLC**

Batavenweg 7

5349BC Oss, NL



MADE IN USA

BOSLAB-147

0316PR272