

# Bosworth Duz-All®

## PMMA All Purpose Self Cure Acrylic Repair Material

Duz-All® is a self-curing acrylic intended for any denture emergency involving broken teeth, repairing fractures, adding clasps, making cold cure full dentures, rebasing or correcting full or partial dentures, adding a post dam and it can be used for making custom impression trays, repairing or making jacket crowns as well as making orthodontic appliances.

Duz-All® sets in less than 20 minutes with no special equipment required and it is available in pink, white, clear and veined pink powder.



### GENERAL MIXING INSTRUCTIONS

- 1 Place a small amount of Duz-All® powder in a jar. The amount of powder required will depend on the repair or other application and the size.
- 2 Add the Duz-All® liquid with an eye dropper. Use sufficient liquid to moisten the powder creating a heavy creamy consistency.  
**RATIO**  
2-3 parts powder to 1 part liquid by volume.
- 3 Immediately mix vigorously with a clean spatula for about ½ minute. Cover the mixing jar to prevent evaporation of the liquid. Allow the mixture to set for about 2 and ½ minutes.
- 4 While the material is setting take a brush and apply the Duz-All® liquid to the area where the material is to be applied. This is necessary to create a cohesive bond.
- 5 Within 10 minutes the material will radiate sufficient heat to complete the required chemical cure.
- 6 Finish and polish.

**Caution: Certain individuals may be sensitive to products containing acrylic.**

*This product, like many other self-cure acrylics, generates heat during polymerization. Caution should be exercised by dentists and/or technicians to insure material is not left in the patient's mouth or on hands during polymerization. Use this product in a well ventilated area wearing proper clothing and eyewear.*

### ORDERING INFORMATION

166260	Standard Kit (Specify Powder Shade)
	 
	8oz (226g) Powder      4oz (118 ml) Liquid
166261	8 oz (226g) Powder Only (Specify Shade)
166262	4 oz (118 ml) Liquid Only
166264	1 lb (454 g) Powder Only (Specify Shade)
166265	8 oz (236 ml) Liquid Only
166269	32 oz (946 ml) Liquid
166267	5 lb (2.3 kg) Pink Powder
166267C	5 lb (2.3 kg) Clear Powder
166267V	5 lb (2.3 kg) Vein Powder

BULK PACKAGING AVAILABLE.

## Bosworth® Company

Please forward health & safety questions to:  
**The Harry J. Bosworth Company**  
A division of Keystone Industries  
52 West King St.  
Myerstown, PA 17067  
800.333.3131  
www.keystoneind.com

EC REP

Keystone Europe LLC  
Batavenweg 7  
5349BC Oss, NL



MADE IN USA

BOSLAB-175

0516PR119