


## Bosworth Trim Plus Instructions for Use

Trim Plus is a fast-setting, non-irritating PMMA temporary crown and bridge material formulated for extended use.


### DIRECTIONS




1. Take an impression with material of choice. (Recommended: Bosworth Supergel)

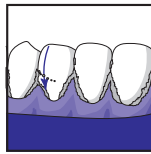


2. Complete tooth preparations. Take another impression from which the final crown or bridge is to be made. Match tooth shade to closest Trim Plus shade.

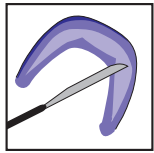


3. Cut peripheral border and interproximal areas for best fit.

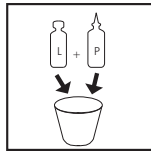


4. Mix powder and liquid until honey-like consistency. Approximate powder to liquid ratio is 6.3 g of powder to 3 ml of liquid. This amount would make 9 full jacket crowns.

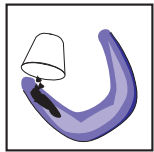


5. Fill each tooth area within the impression where preparatory work has been done with Trim Plus acrylic.

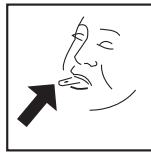

6. When Trim Plus gets dull, reseat the impression to return to its original position.

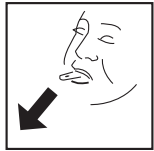

7. Leave in place for 30-90 seconds. Remove when first feels warm. When material is rubbery it may be removed from impression and trimmed. May be placed back into impression until fully set.

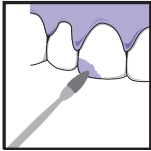

8. Trim and polish as desired. Cement bridge or crown in place with a non-eugenol cement.

### NOTES

- Using Bosworth Gorilla Seal or Glaze saves polishing time. Gorilla Seal will bond directly to the temporary when cured with a halogen bulb.
- Do not allow material to cure in mouth.
- May be bench cured or placed in pressure pot (Recommend: Bosworth Pressure Pot).

CAUTION: Certain individuals may be sensitive to products containing acrylic.


WORKING TIME: 3-4 minutes TOTAL SET TIME: 10 minutes

**Bosworth Company**

Polymethyl Methacrylate Temporary Resin Acrylic

**Bosworth® Trim Plus®**

### ORDERING INFORMATION

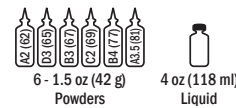

REF 0921930 Standard Kit

0921931	1.5 oz (42 g) Powder Select Shade
0921931s	1.5 oz (42 g) Fast Set Powder Select Shade
0921936	4 oz (118 ml) Liquid Only
0921933	1 lb (454 g) Bulk Powder, Select Shade
0921933s	1 lb (454 g) Bulk Fast Set Powder, Select Shade
0921937	8 oz (236 ml) Liquid Only

ADDITIONAL BULK PACKAGING AVAILABLE.

### TOOTH SHADE CONVERSIONS

BOSWORTH	VITA	NEW HUE
59	B1	59
60	A1	60
61	B2	61
62	A2	62
65	D3	65
66	A3	66
67	B3	67
68	--	68
69	C2	69
77	B4	77
81	A3.5	81
87	A4	87
	C3	
	C4	
	D4	

### AVAILABLE POWDER SHADES:

- B1 (59)     B3 (67)  
 A1 (60)     68  
 B2 (61)     C2 (69)  
 A2 (62)     B4 (77)  
 D3 (65)     A3.5 (81)  
 A3 (66)     A4 (87)  
 Incisal


**Bosworth Company**

Please forward health & safety questions to:

The Bosworth Company  
7227 North Hamlin Avenue  
Skokie, IL 60076-3999 USA or  
F. Moore

Unit 3, Haldene Way  
Seaham Grange Ind. Estate, Seaham  
CO Durham, SR7 OPU England  
www.bosworth.com • hjinfo@bosworth.com

Flashpoint 50°F (10°C)

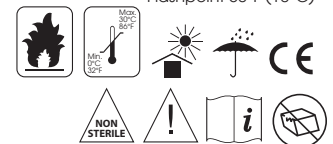

Made in USA  
0813PR166