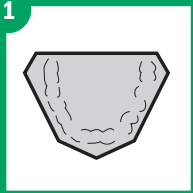
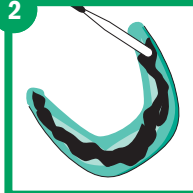
Bosworth TruRepair® mode d'emploi

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


Bosworth TruRepair® Instructions for Use

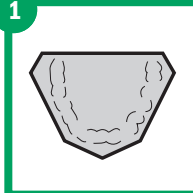
TruRepair is a fast-bonding acrylic repair material for repairing fractured dentures, replacing broken or dislodged denture teeth, and any additional office or lab emergency. A special internally-pigmented spherical globe of powder will blend with denture base materials to produce a color-stable match. TruRepair's self-curing liquid and powder system produces a cross-linked, high-strength bond. Powder is available in pink, vein, and clear.

- 

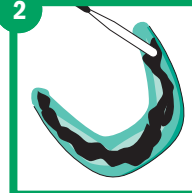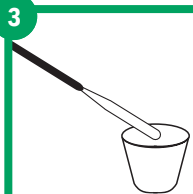Examine the denture: Bevel the edge of the fracture area and bur down to fresh acrylic where the repair is to be made.
- 

Use alginate or PVS material as a repair matrix. Place denture in matrix. Allow to set.
- 

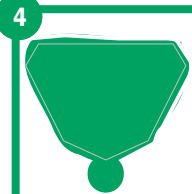Mix TruRepair.

RATIO
2-3 parts to 1 part liquid by volume.
- 

Apply TruRepair to denture fracture or chip, over fill allowing for room to bur down to a naturally fitting denture. Vibrate if needed.
- 

Bench cure 10-15 minutes and remove from matrix. Finish by using **Bosworth Hi-Spot Indicator Paste** for a perfect fit and bur to polish.

Bosworth Company

Material de reparación acrílico de adhesión rápida
Material de reparo acrílico de adhesão rápida
Materiale acrilico per riparazioni a presa rapida
Schnell abbindendes Acryl-Reparaturmaterial
Matériau de réparation acrylique à prise rapide

Fast Bonding Acrylic Repair Material

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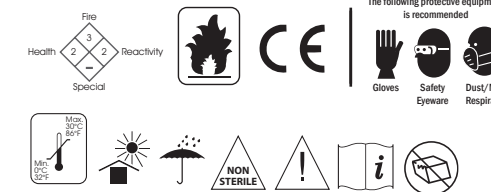
BULK PACKAGING AVAILABLE INCLUDING
5 LB POWDER AND 1 QT LIQUID KIT.

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