

# Original Truliner®

**Original Truliner® is a color stable, permanent PMMA denture corrective reline material formulated for extended life (3-5 years).**

The extremely fine-milled powder cures to a dense consistency and prevents the reline from assuming odors, tastes, or stains. Original Truliner is a unique reverse curing acrylic (cures from the inside out) which releases the free monomer before it is inserted in the mouth, eliminating chemical and thermal sensitivity and forming a permanent bond in as little as ten minutes that will not chip or discolor. Can also be used for repairs and orthodontic appliances.

Mixing Ratio: 7.6 ml liquid (1 full glass vial) to 15 ml powder (1 full plastic vial).

Tips: Before making a permanent denture reline, treat the tissue with a tissue conditioner such as Bosworth Softone to eliminate the possibility of the final corrective reline duplicating the disturbed or deformed mucosa caused by the deficient denture.

Note: New Truliner liquid and powder is NOT interchangeable with Original Truliner liquid and powder. New Truliner is a PEMA formula and Original Truliner is a PMMA formula.

Recommended for use with: Cora-Caine®, Glaze™, Masque™, and Pressure Pot



## Standard Kit ( 8oz. Powder, 6 oz. Liquid)

PRODUCT	SIZE	ITEM NUMBER
Standard Kit, Pink	8 oz. (227g)	0921940
Standard Kit, Vein	8 oz. (227g)	0921941
Standard Kit, Clear	8 oz. (227g)	0921942

## Components

PRODUCT	SIZE	ITEM NUMBER
Powder, Pink	8 oz. (227g)	0921943
Powder, Vein	8 oz. (227g)	0921944
Powder, Clear	8 oz. (227g)	0921945
Liquid	6 oz. (177g)	0921946



[dental.keystoneindustries.com](http://dental.keystoneindustries.com)

US: 480 South Democrat Road Gibbstown NJ, 08027  
EU: Keystone Industries GmbH Stockholzstr. 11 78224 Singen, Germany

US: 1.800.333.3131  
EU: +49 77 31 91 21 01