
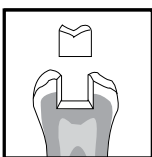
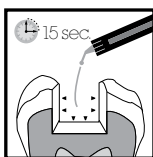
Bosworth Acid Etch Instructions for Use


1. Prepare the tooth to the desired configuration.

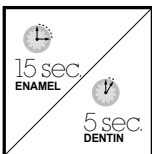
2. Attach Flowthru® Microbrush®-style tip to the etchant syringe and dispense etchant gel either directly onto the tooth surface or onto pad and brush onto tooth surface. The etchant should be placed first on enamel surfaces, then on the dentin.

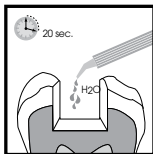3. Recommended etching time for enamel is 10-15 seconds. An adequate etch of the dentin can be achieved in 5 seconds.

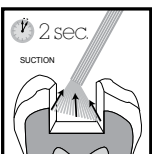4. Rinse well with water for 10-20 seconds removing all of the etchant material.


5. Remove all excess moisture by placing high volume suction over the prepared surfaces for 2 seconds. The prepared tooth surface should be visibly moist, but not desiccated or saturated with water.

WARNING: If Acid Etch contacts skin or soft tissue, rinse off immediately with large amounts of water.


CAUTION

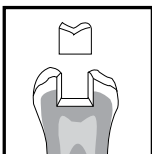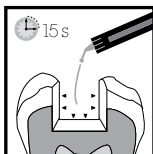1. Acid Etch contains phosphoric acid which may cause burns or skin sensitization in susceptible individuals.
2. Before using this product cover the patient's eyes with protective eyewear.
3. Avoid contact with oral tissues, eyes and skin. If Acid Etch contacts eyes, skin or soft tissue, rinse off immediately with large amounts of water while using an oral evacuator or saliva ejector and seek medical attention.
4. Acid Etch should extrude easily from the syringe. DO NOT USE EXCESSIVE FORCE. Excessive pressure may result in unanticipated extrusion of the material.
5. Do not use Acid Etch on dental pulp tissue; cover dentin close to the pulp (< 1 mm remaining) with a hard setting calcium hydroxide liner (Bosworth Hydrox).

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