
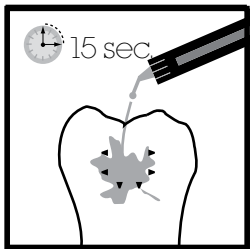
Bosworth Aegis® Pit & Fissure Sealant Instructions For Use

AEGIS PIT & FISSURE SEALANT is an opaque 38% filled light-cured resin with Amorphous Calcium Phosphate (ACP). The sealant cures to an opaque film on enamel. ACP will provide a source of calcium and phosphate ions.


BEFORE USING PIT & FISSURE SEALANT: Clean teeth to be sealed thoroughly using your favorite method. (Air abrasion or a thin diamond bur may be used to clean fissures.) If necessary, use a rubber dam to isolate teeth.

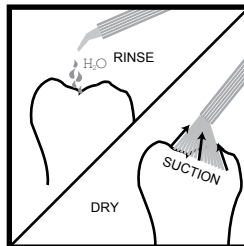
1. Apply BOSWORTH ACID ETCH for 15 to 20 seconds on enamel.

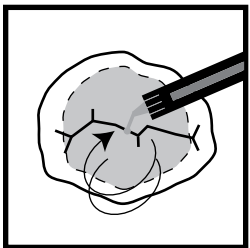2. Rinse ACID ETCH off with water. Dry with suction or clean, oil-free air.

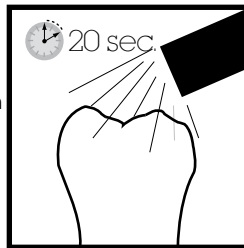3. Apply AEGIS PIT & FISSURE SEALANT directly with the disposable application tip or use a disposable brush.

4. Cure AEGIS PIT & FISSURE SEALANT with light for 20 seconds.* Final Set time is 30-40 seconds.

NOTE: After each use dispose of dispensing tip and recap syringe. Prior to next use, attach a new disposable tip.

***CURING TIMES:** Use a standard high-intensity curing light (intensity greater than 300 mWatt/cm² in the wavelength range of 300-500 nm) . Lower powered lights can require longer curing times while very high intensity lights can cure in shorter times.

CAUTIONS

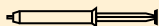
1. Do not use AEGIS PIT & FISSURE SEALANT on persons with a sensitivity to acrylics.
2. Wear protective gloves and eye protection when using this product.
3. See SDS for safe handling precautions and proper waste disposal. (www.keystoneindustries.com/en/msds)

ORDERING INFORMATION

INFORMATIONS CONCERNANT LES COMMANDES • BESTELLINFORMATIONEN • INFORMAZIONI PER LE ORDINAZIONI
INFORMAÇÃO PARA ENCOMENDA • INFORMACIÓN DE PEDIDOS

3 - 1.2ml syringes
Aegis P&F Sealant

1 - 3ml syringe
Acid Etch

REF 0921348

Standard Kit

Kit standard

Standardsatz

Kit standard

Kit standard

Kit estándar

Box of 3 - 1.2ml Pit & Fissure Syringes
and 1 - 3ml Acid Etch Syringes

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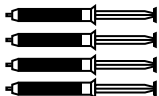
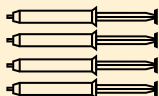
4 - 3ml syringes Acid Etch
Phosphoric Acid Syringes

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