

Enamelite® CAD/CAM Scanning Spray

Glaze Ceramics Like Never Before

Glazing dental ceramics has become faster and easier since Enamelite spray glazes took the task head-on.

One of the newer Enamelite® sprays available, our CAD/CAM Scanning Spray was created with detailed accuracy in mind. The spray offers features that other materials aren't able to match, such as covering materials with the thinnest, most uniform and accurate layer. This allows for a more accurate scan, and in turn, a more accurate fit. All of these features have been made possible through Enamelite's patented aerosol packaging technology used across the entire line of dental ceramic sprays, making glazing a fast and easy process.

The CAD/CAM Scanning Spray is meant to be sprayed prior to scanning, and should only be used for non-intraoral scanning, such as on crowns, models, and impressions.




FEATURES AND BENEFITS

- Stains and glazes multiple units in one firing
- No puddling at margins
- Works on all ceramic restorations
- One can can be used for hundreds of applications
- Use on any ceramic crowns for optimum results
- Improves esthetics in multiple lighting conditions

ORDERING INFORMATION

88g/3.1 oz. cans

#5100176

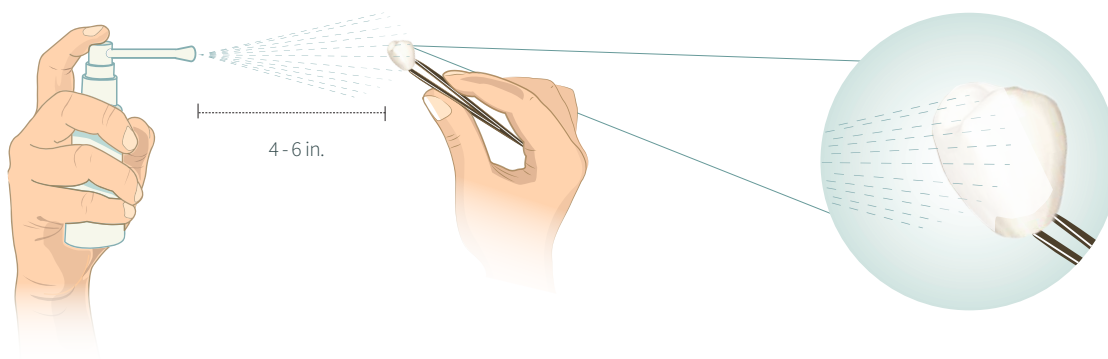

Hold glaze can using nozzle extension approximately 4 to 6 inches from the restoration. Use short bursts around restoration.

A nice even coating of white should appear. Make sure there is no puddling of the glaze.

keystone®
i n d u s t r i e s



keystone
industries

480 South Democrat Rd. Gibbstown, NJ

About Us: The Keystone Mission

Since 1908, Keystone Industries has maintained a strong reputation of producing innovative high-tech dental products in both the operatory and laboratory realms. We are committed to the goal of total customer satisfaction. We are dedicated to the achievement of this goal by meeting or exceeding our customer expectations for product quality, competitive pricing and prompt delivery and customer service.



Video Learning Center

Our new "In The Know" video series gives you an up close look at hands-on demonstrations of products being done by professionals in the field.



Product Literature

Download all catalogs, brochures, booklets, sell sheets, SDS, and instructions from our online databases.




Promotions


We always have our quarterly promotions posted on our website. Check them out and take advantage of the best dental deals in the industry.


Contact Information

Domestic Office

480 South Democrat Road
Gibbstown NJ 08027

 800-333-3131


 856-663-4700


 856-224-9444

 customerservice@keystoneind.com

International Office

Werner-von-Siemens Str. 14a,
D-78224 Singen, Germany

 +49 77 31 91 21 01

 +49 77 31 91 21 02

 sales@keystoneind.com