

Enamelite[®] Low-Fusing Fluorescent Spray Glaze

Glaze Ceramics Like Never Before

Glazing dental ceramics has become faster and easier since Enamelite spray glazes took the task head-on.

Staining and glazing all ceramic units can be a laborious task when using certain glazes. Many of these ceramic glazes have to be fired multiple times for the unit to glaze out with optimum results. With Enamelite Fluorescent Low Fusing Glaze, this can be accomplished in one firing and with outstanding results.

With fluorescence built into the glaze, you achieve results that help eliminate negative results in certain lighting conditions. This product is perfect for: Full Contour Zirconia Crowns-eMax-All on Four Zirconia Bridges.



Zirconia Bridge


FEATURES AND BENEFITS

- Stains and glazes multiple units in one firing
- No puddling at margins
- Works on all ceramic restorations
- One can can be used for hundreds of applications
- Use on any ceramic crowns for optimum results
- Improves esthetics in multiple lighting conditions

ORDERING INFORMATION

46g/1.6oz can

#5100181

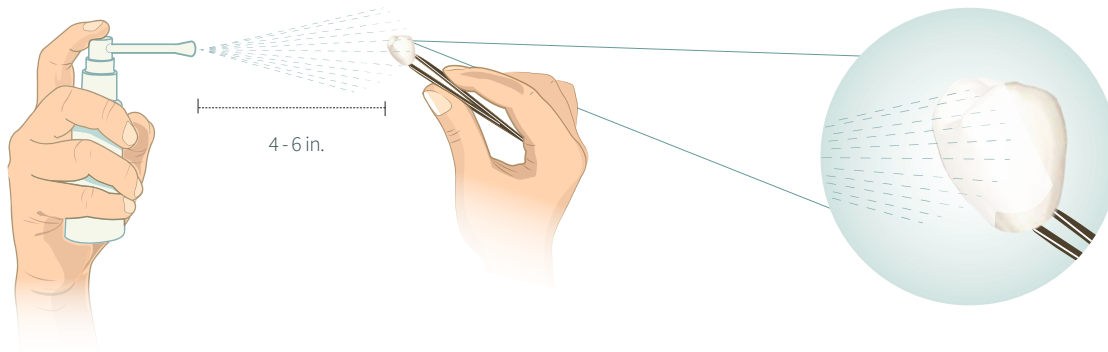

Hold glaze can using nozzle extension approximately 4 to 6 inches from the restoration. Use short bursts around restoration.

A nice even coating of white should appear. Make sure there is no puddling of the glaze.

keystone
industries

US: 1.800.333.3131 | EU: +49 77 31 91 21 01 | keystoneindustries.com