

Itsoclear Clasps

Crystal Clear Quality

Itsoclear provides the ability to produce and repair a clasp in less than 3 minutes with cosmetic appeal unlike any other clasp.

Use Itsoclear clasps for: repairs, acrylic partials, cast partials with clear clasps in less than 3 minutes, Replace metal clasps on existing partials, Acrylic Nesbit with clear clasps.

FAQs

What's the learning curve of the product?

A person with no experience can be taught how to use Itsoclear in one day. There's minimal training for lab tech.

How do you prepare the repair to replace a clasp on an old partial?

Cut off the clasp, notch tooth and acrylic on lingual. Heat clasp into notched area.

How are clasps attached to the partials?

First create a loop with the tail of the clasps for retention, then embed and cover tail with acrylic.

What happens if you overheat the clasp?

It will bubble, become brittle and break.

What is the most important thing to keep in mind with adapting a clasp?

Do NOT overheat the clasp.

How are black spots avoided in the mouth?

When applying acrylic, make sure you extend onto the inner proximal area.



FEATURES AND BENEFITS

- Crystal clear, will not lose clarity
- Non-abrasive
- Easily adjustable
- No injection machines or hand-held injectors required
- No installation needed
- Perfect for new partial dentures, as well as repairs & replacing metal clasps
- Saves bench time while producing a higher quality dental aesthetic clasps which customers will appreciate & increase profit margins

ORDERING INFORMATION (5 tubes per package)

Universal Clasps – 1.8mm thick – 6/Pkg	#1009150
Molar Clasps – 2.2mm thick – 6/Pkg	#1009151
Universal Clasps – 1.8mm thick – 36/Pkg	#1009152
Molar Clasps – 2.2mm thick – 36/Pkg	#1009153