

Support Peg

USE FOR CROWNS & VENEERS

1. Using the Support Peg syringe, simply fill the crown, jacket, or veneer; no drying time is necessary.
2. The crown, jacket, or veneer is easily balanced on the ceramic peg and does not tilt during the firing. Support Peg offers firing distortion for temperatures from 400° to 1300° C.
3. Support Peg used on a Veneer.
4. Veneer on Pink Peg.
5. Simply remove with a slight twist from the ceramic peg after firing and tap off any residual Support Peg material.

