

PRO-FORM®

Bleaching Laminates

Proform bleaching laminates are the ideal foam lined laminates used for dental bleaching. It's unique foam design allows for better contact of the bleaching treatment with the enamel, resulting in less time required for bleach application.

Instructions for Use:

- 1) Remove the protective coating layer from the hard side of the bleaching laminate.
- 2) Place the laminate into the vacuum unit with the foam facing down towards the model.
- 3) Use a model trimmer to reduce the model base height within 5 mm of the gingival line.
- 4) Heat the material 20-30 seconds until the foam gets very soft. NOTE: this material does not droop. If the material turns yellow or brown, it has been overheated.
- 5) Use crown & bridge scissors to remove the material from model. Trim appliance along the gingival line.
- 6) Bleaching material is placed into the finished tray. The filled tray is placed into the patient's mouth according to bleach manufacturer's instructions for use.