

Hatho Thermopol Set

Less work for better results is what the Hatho ThermoPol Set is built upon.





About Hatho ThermoPol Set

It's revolutionary design and working technique will brighten not just acrylics, but also thermoplastics such as Valplast® and TCS®. This all-in-one kit is the only of its kind on the market, and will take your denture polishing work to a whole new level.



PRODUCT	ITEM NUMBER
Hatho ThermoPol Set	1670540

HATHO THERMOPOL SET INCLUDES:

	2x		1670540	HATHO MULTI LAYER BRUSH
	1x	90 g	1670542	HATHO POLISTAR LINTGREY COMPOUND
	1x	100/14	1670543	HATHO COSIMA SOFT BUFF
	1x	90 g	1670544	HATHO POLISTAR CREAM
	1x	150 g	1670546	HATHO POLISTAR PRO PINK
	1x	12/PKG	1670547	HATHO ALPHACARD POCKET CARD
	1x	4" x 50 H	1670523	HATHO MIRA POLISHING BUFF



3 Steps Thermoplastic Polishing

1st Step: Pre-polishing of thermoplastic dentures with
HATHO Abrasive Trimming Wheel and HATHO Polistar Lintygrey compound



+



POLISTAR LINTYGREY 90G

1,400 rpm



2nd Step: Shine polishing of thermoplastic dentures with
HATHO COSMINA Soft buff and HATHO Polistar Cream



100/14

+



POLISTAR CREAM 75G

1,400 rpm



3rd Step: High-shine polishing of thermoplastic dentures with
HATHO Mira buff and HATHO Polistar Pro Pink



4" X 50 H

+



POLISTAR PRO PINK 150G

1,400 rpm



keystone[®]
industries

dental.keystoneindustries.com

US: 480 South Democrat Road Gibbstown NJ, 08027
EU: Keystone Industries GmbH Stockholzstr. 11 78224 Singen, Germany

US: 1.800.333.3131
EU: +49 77 31 91 21 01