



Z-Bur

The special hollowed, fluted design of the Z-Bur Diamond Bur is perfect for easy removal of sticky tissue conditioners and pastes commonly used to take denture impressions.

About the Z-Bur

The Z-Bur Diamond Bur also works well in removing most soft liners. Z-Bur action set at a medium speed (15,000-25,000 rpm) eliminates messy material removal. The slower cutting speed reduces liner burning and melting. The hollowed design also reduces the carbide clogging and minimizes flying debris. Once the liner is removed, the final hard surface can be prepared with the carbide of choice.

Features & Benefits

- Reduces burning and melting
- Reduces debris
- Sold individually



PRODUCT	QUANTITY	ITEM NUMBER
Z-Bur	100/Pkg	1201995



dental.keystoneindustries.com

US: 480 South Democrat Road Gibbstown NJ, 08027
 EU: Keystone Industries GmbH Stockholzstr. 11 78224 Singen, Germany

US: 1.800.333.3131
 EU: +49 77 31 91 21 01