

# Vibrator

**Instructions for Use**

# Instructions for Use

1. Make sure switch is in extreme left (off) position. Plug cord into power outlet.
2. To activate unit, turn switch knob in clock-wise direction for desired amount of vibration.
3. Low vibration is recommended for delicate and precise bridges.
4. Medium or high vibration is recommended for stone and plaster model and investment work.
5. It is recommended that the unit be turned off when the vibrator is not being used. This will ensure maximum trouble free service.
6. To clean, wipe with a damp cloth after use and before investing material is set.
7. Unit is warranted for its designated use, in accordance with the instruction, against original defects in material and workmanship for a period of 12 months from date of purchase.
8. Warranty is void if any repair is attempted by anyone other than the manufacturer.
9. Return unit to dealer for service.