

# +OPTIMIZE

INSTANT FOG FREE

**ELIMINATES WATER &  
FOG DISTORTION**  
**FORMULATED FOR USE WITH  
HIGH SPEED HAND PIECES**


Mint Fragrance



Keep out of reach of children.

Net Contents:  
32 fl oz (946 ml)

REF 00118

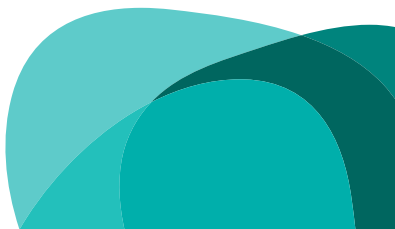

- Ready to use solution
- Prevents fogging and distortion of dental mirrors



**Directions:**

1. Dispense 1 fl oz (29.57 ml) of solution into a small disposable cup that can accommodate a dental mirror.
2. Dip dental mirror into solution and shake off excess solution.
3. If necessary, reapply solution during use to prevent fogging or distortion.
4. Discard the solution and disposable cup after use with each patient.

**DO NOT REUSE THE SOLUTION BETWEEN PATIENTS**

Do not take internally. Solution is preserved with less than 1% benzylkonium chloride.  
Refer to product SDS for additional health, safety and disposal information.

Keystone Industries  
52 West King St.  
Myerstown, PA 17067 USA  
800.333.3131  
keystoneindustries.com

**Made in USA**

keystoneindustries.com/symbolglossary

**International Sales**  
Keystone Industries GmbH,  
Stockholzstr. 11, 78224 Singen, Germany  
☎ +49 77 31 91 21 01

EPR-17 NatLab-1361 4926643 02/2021