



+OPTIMIZE

SUPER-KLEEN 40

EVACUATION CLEANER

POWDER CONCENTRATE


Cleaner & Deodorizer For
Evacuation Equipment



Keep out of reach of children.

Net Contents: 2 lbs (907 g)
Makes 32 gallons (121.13 L)
Approximately 80 cleanings

REF 00136

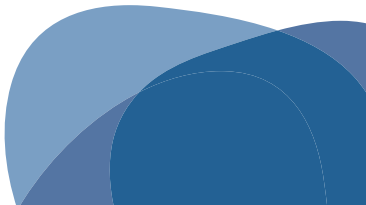

Concentrated evacuation cleaner. Powerful, non-foaming, pleasant scent.

Directions: Add 2 cap fulls (approx. 1 oz, 20.57 ml) of Optimize Super-Kleen powder to 1 gallon (3.78 L) of warm water. Shake or stir well to dissolve. You are now ready to pour Optimize Super-Kleen into your dental evacuation system.

At the end of each working day, flush 1 quart (32 fl oz, 946 ml) of Optimize Super-Kleen through all suction lines. Run system for 5 minutes to clear lines. For high use operatories, flush evacuation lines twice per day. Collection canister should be cleaned weekly with brush and Optimize Super-Kleen solution.

Conforms to EPA rule for Amalgam Separators.



Caution: Wear protective gloves.

Warning

Causes skin irritation. Causes serious eye irritation. May cause respiratory irritation if powder inhaled. May be harmful if swallowed. Refer to product SDS for additional health, safety and disposal information.



Keystone Industries
52 West King St.
Myerstown, PA 17067 USA
800.333.3131
keystoneindustries.com

International Sales

Keystone Industries GmbH,
Stockholzstr. 11, 78224 Singen, Germany
+49 77 31 91 21 01

Made in USA

keystoneindustries.com/symbolglossary
EPR-Q3 NatLab-1357 4926637 02/2021