

+ OPTIMIZE

SUPER-VAC 40 PLUS

EVACUATION CLEANER

LIQUID CONCENTRATE

Mint Fragrance

Net Contents:
64 fl oz (1.89 L)

Makes 8 gallons (30.28 L)
1:15 dilution ratio

REF 00138



EPA-Compliant. Safe for use with amalgam separators.

- Multi-Enzyme System
- Non-Foaming Formula
- Deodorizes as it cleans
- Dissolves organic fluids and debris
- Environmentally friendly, neutral pH formula
- Daily use improves cleaning efficiency



Directions:

Dilute 2 fl oz (59.14 ml) of Optimize Super-Vac 40 Plus with 30 fl oz (887.20 ml) of warm water. Mix well and flush through evacuation system, high volume, and saliva ejection hoses.

First Aid:

In case of contact, immediately flush eyes or skin with plenty of water for at least 15 min. If solution is swallowed or comes in contact with eyes, immediately seek medical attention.

Danger

May cause skin irritation. May cause eye irritation. May cause allergy or asthma symptoms or breathing difficulties if inhaled. Refer to product SDS for additional health, safety and disposal information.



Keystone Industries
52 West King St.
Myerstown, PA 17067 USA
800.333.3131
keystoneindustries.com

International Sales

Keystone Industries GmbH,
Stockholzstr. 11, 78224 Singen, Germany
+49 77 31 91 21 01

Made in USA

keystoneindustries.com/symbolglossary

EPR04 NATLAB-1355 4926642 02/2021