

+ OPTIMIZE

ULTRASONIC SOLUTION

TARTAR, STAIN & PERMANENT CEMENT REMOVER LIQUID



Keep out of reach of children.

Net Contents:
128 fl oz (3.78 L)

REF 00139

CAUTION: US Federal law restricts this device to sale by, or on the order of a dental professional.



Quickly removes food stains, calculus deposits, tartar build-up and tobacco related stains from appliances such as dentures, partials, or removable orthodontics.

NO MIXING NECESSARY. DO NOT DILUTE.

Strong acid solution. Handle with care. Thoroughly rinse all items with tap water after removal from Tartar & Stain Solution. To prevent irritation of gingival tissue, place prosthetic in a solution of 1 tsp baking soda to 8 oz water for 30-60 seconds.

Directions:

1. Place items in glass beaker, cover items with full strength solution. Caution: 1-2 minutes maximum cleaning time on all instruments. Instruments will discolor if left in solution longer than recommended. Bridges, dentures, gold casting, orthodontic appliances (5-10 minutes) Acrylic splints, crowns (5 min).
2. Put glass beaker in tank containing Optimize **General Purpose Cleaning Solution** for sound conduction and to neutralize any **Tartar & Stain Liquid** that might accidentally be spilled inside.



Caution: Strong acid solution can cause irritation upon contact with skin. Wear protective gloves/protective clothing/eye protection. Immediately flush eyes with water for 15 minutes upon contact.

Danger

Causes severe skin burns and eye damage. Harmful if swallowed. Refer to product SDS for additional health, safety and disposal information.



Keystone Industries
52 West King St.
Myerstown, PA 17067 USA
800.333.3131
keystoneindustries.com

International Sales

Keystone Industries GmbH,
Stockholzstr. 11, 78224 Singen, Germany
+49 77 31 91 21 01

Made in USA

keystoneindustries.com/symbolglossary

EPR-39 NATLAB-1351 4926628 02/2021