

+ OPTIMIZE

READY TO USE SOLUTION


TEMPORARY CEMENT REMOVER LIQUID



Keep out of reach of children.

Net Contents:
128 fl oz (3.78 L)

REF 00140

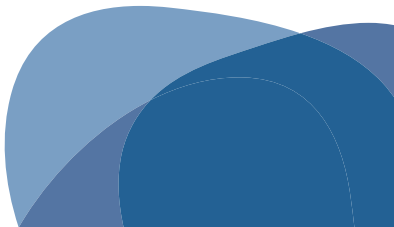

Strong solution to remove temporary cement.

For use prior to final sterilization. Do not dilute.

Directions:

1. Total immersion time for items not to exceed 30 minutes.
2. Rinse items thoroughly with water prior to sterilization.
3. Refresh solution daily or when solution becomes cloudy.



Caution: Strong alkaline solution. Wear protected gloves/protected clothing/eye protection. Can cause serious irritation upon contact with skin and eyes. Immediately flush skin and/or eyes with copious (large) amounts of water upon contact. Seek medical attention as needed.

Danger

Causes serious eye irritation. Refer to product SDS for additional health, safety and disposal information.



Keystone Industries
52 West King St. Myerstown, PA 17067 USA
800.333.3131
keystoneindustries.com

International Sales

Keystone Industries GmbH,
Stockholzstr. 11, 78224 Singen, Germany
+49 77 31 91 21 01

Made in USA

keystoneindustries.com/symbolglossary
EPR 44 NATLAB-1352 4926641
02/2021