

+OPTIMIZE

TRAY CLEANER


**CLEANS METAL
IMPRESSION TRAYS**

POWDER CONCENTRATE

Removes ZOE/alginate from impression trays.

Net Contents:
16 oz (454 g)
Makes 4 gallons (15.15 L)

REF 00141

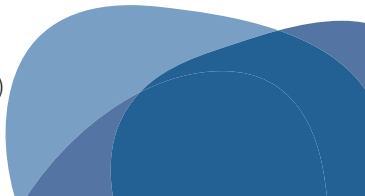

Directions: Add 2 cap fulls (approximately 1 oz, 30 g) of **Tray Cleaner** to one quart (32 fl oz, 946 ml) of warm water. Mix the solution well. Place trays and any other material with alginate, wax or compound residue into the water. Remove as much alginate, wax or compound as possible beforehand by scraping with a spatula. Maximum immersion time should be 4-6 hours. Rinse trays thoroughly with water prior to sterilization. Fresh solution should be made daily.



Warning

Causes skin irritation. Causes serious eye irritation. May cause respiratory irritation. May be harmful if swallowed. Refer to product SDS for additional health, safety and disposal information.



Keystone Industries
52 West King St.
Myerstown, PA 17067 USA
800.333.3131
keystoneindustries.com

International Sales

Keystone Industries GmbH,
Stockholzstr. 11, 78224 Singen, Germany
+49 77 31 91 21 01

Made in USA

keystoneindustries.com/symbolglossary EPR10 NatLab-1356 4926636 02/2021