

# ***Millennium*** **Pour Denture Powder**

**Type 2, Class 2**  
**Net Content: 1 lb.**

# Millennium Pour Denture Powder

**Instructions:** Use 2 parts pour powder to 1 part pour liquid. Mix for 15 seconds, making sure all polymer beads are wetted out. Pour immediately into sprue lead of the pour flask, taking care to not incorporate air into the acrylic mixture. Submerge 3/4 of the pour flask into a pressure pot filled with preheated water, 115°F +/- 5°F (47°C +/- 3°C) at 15 - 25 PSI for a minimum of 30 minutes. De-flask and finish case in usual manner.



## Warning

May form combustible dust concentrations in air. May cause an allergic skin reaction. Handling and/or processing of this material may generate a dust which can cause mechanical irritation of the eyes, skin, nose and throat. Due to the spherical nature of this product, spills should be cleaned up promptly to prevent slips and falls. Refer to product SDS for additional health and safety information.



Warning: This product can expose you to chemicals including titanium dioxide, which is known to the State of California to cause cancer. For more information go to [www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov).

Rx Only  



52 West King Street  
Myerstown, PA 17067 USA  
800-333-3131  
[keystoneindustries.com](http://keystoneindustries.com)

EC REP

Keystone Europe LLC  
Batavenweg 7  
5349BC Oss, NL 

[keystoneindustries.com/symbolglossary](http://keystoneindustries.com/symbolglossary)

NATLAB-0280 06/2019