Bosworth Fastray®

Custom Tray and Acrylic Base Plate Material

Bosworth Fastray® **mode d'emploi**
Bosworth Fastray® **Gebrauchsanleitung**
Bosworth Fastray® **Istruzioni per l'uso**
Bosworth Fastray® **Instruções de uso**
Bosworth Fastray® **instrucciones de uso**

Porte-empreinte individuel et matériau pour plaque-base acrylique
Autopolymerisat für individuelle Abformlöffel und Kunststoff-Basisplatten
Materiale per portaimpronte individuali e per basette in resina acrilica
Material para placas de base acrílicas e moldeiras individuais
Material para cubetas individuales y bases acrílicas

 Keystone Industries
52 West King St.
Myerstown, PA 17067 USA
800.333.3131
keystoneindustries.com

Flashpoint 10°C (50°F)

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

keystoneindustries.com/symbolglossary

Autopolymerizable Powder & Liquid

PR121 BOSLAB-234 REV 1 06/2021


Bosworth Fastray Instructions for Use

Fastray is a dimensionally stable, radiopaque, self-curing individual tray and acrylic base plate material in regular (6 minute) or extra-fast (5 minute) set. Fastray is available in blue, white and pink powder.


DIRECTIONS

1. Soak model in 72°F (22°C) water for 3-4 minutes. Prepare model with separating medium.*1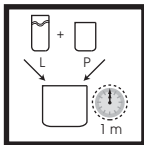

2. Add one full measure (1 oz., 29g) of powder to one full measure (6ml) of liquid and mix for one minute.*2

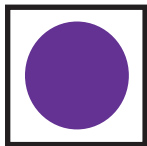3. Knead with wet fingers for 30 seconds. Form into a 2-inch round wafer for an upper tray or into a U-shaped wafer for a lower tray.*3


4. Place wafer on model using light pressure with fingers until an even thickness is formed.

5. Trim excess with a knife and use to form a handle. Press into position.

6. Use a blunt instrument to make perforations on the palate.

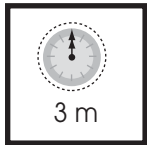7. Bench cure for 3 minutes.*4

8. Trim and polish as desired with acrylic bur.

NOTES

1. A separator such as Bosworth Masque is recommended to prevent product from sticking to the model or fingers.
2. Higher temperature will accelerate the set. Lower temperature will decelerate the set.
3. Place wafer between two wet glass slabs and squeeze into desired thickness for a base plate.
4. To accelerate cure, place in warm water. If wax has been used for undercuts, place in cold water to prevent melting and distortion.
5. Bosworth Rimseal is available to make peripheral impressions for dentures and border molding.

See SDS for safe handling precautions and proper waste disposal.

(<https://dental.keystoneindustries.com/sds/>)

CAUTION: Certain individuals may be sensitive to products containing acrylic.

This product generates heat during polymerization that could be harmful to the patient or dental professional. Do not cure material in mouth.

WORKING TIME: 3-4 minutes TOTAL SET TIME: Regular 6 min. - Extra-Fast 5 min.

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