

# +OPTIMIZE

## EVACUATION SYSTEM CLEANER PLUS

### EVACUATION SYSTEM CLEANER WITH IMPROVED MULTI-ENZYME BLEND

#### Liquid Concentrate

Peppermint Fragrance

Net Contents:  
128 fl oz (3.78 L)

Makes 32 gallons (121.13 L)  
1:32 dilution ratio

REF 00148



#### Instructions for Use:

1. Dilute 1 fl oz (29.57 ml) of liquid concentrate with 1 quart (32 fl oz, 946 ml) of water.
2. Aspirate the solution through the high speed evacuation line and saliva ejector line.
3. Pour the remainder into the cuspidor.
4. Repeat for each treatment room daily.

- Deodorizes as it cleans
- Dissolves organic fluids and debris
- Neutral pH, meets EPA Amalgam Rule 40 CFR 441
- Non-foaming formulation
- Daily use improves efficiency
- Environmentally friendly



**For Best Results:** Everyday use is recommended. Evacuation line and chair side traps will be free of foreign materials excluding amalgam. Manual cleaning is virtually eliminated.

#### Harmful if swallowed.

Not for human consumption. Avoid contact with eyes, skin and clothing. In case of contact, immediately flush skin with plenty of water. If solution is swallowed or comes in contact with eyes, call physician immediately.

Avoid contact with plastic laminate surfaces.

#### For professional use only.

#### Danger

May cause skin irritation. May cause eye irritation. May cause allergy or asthma symptoms or breathing difficulties if inhaled. Refer to product SDS for additional health, safety and disposal information.



Keystone Industries  
52 West King St. Myerstown, PA 17067 USA  
800.333.3131  
keystoneindustries.com

#### International Sales

Keystone Industries GmbH,  
Stockholzstr. 11, 78224 Singen, Germany  
☎ +49 77 31 91 21 01

**Made in USA**

keystoneindustries.com/symbolglossary  
NATLAB-1382 4926644 02/2021