



MIZZY Z•O•E CEMENT

Mizzy Z•O•E Cement is Zinc Oxide Eugenol Cement with a setting time from 6 - 8 minutes. It is recommended for lining cavities in posterior and anterior teeth (not for use under acrylic filling materials), for temporary cementation and as a sedative base under metallic restorations.

Z•O•E Cement protects the pulp from thermal shock and possible irritation from freshly mixed cements. Cavities lined with Z•O•E Cement may be filled promptly.

DIRECTIONS FOR USE

Dispense one portion from the powder measure to one drop of liquid from the pipette (held perpendicularly). For convenient mixing and cleanliness, Z•O•E mixing pads are available, and preferred to a glass slab. Always use a stainless steel spatula for mixing.

Place the suggested quantities of powder and liquid on the mixing tablet. Divide the powder into 4 equal portions, add one portion at a time and mix each portion until smooth before adding the next. The mix will be complete and rather stiff in 1.5 minutes. Mixing ratio: 1.4 gm powder to 0.4 ml of liquid.

CAUTION - Do not draw the liquid into the rubber bulb of the pipette as the liquid will be contaminated by the rubber.

 Keystone Industries
52 West King Street
Myerstown, PA 17067 USA
800.333.3131
keystoneindustries.com

 EC REP
Keystone Europe LLC
Batavenweg 7 
5349BC Oss, NL

Report any serious incident occurring with this device to the manufacturer and applicable Competent Authority of the member state in which the user/patient is established.

 MD Medical Device

keystoneindustries.com/symbolglossary
6071505 NATLAB-0168 04/2020