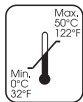
- Ⓔ EN Enamelite Spray Glaze Instructions
- Ⓔ NL Enamelite glazuurspray instructies
- Ⓔ FR Instructions D'utilisation De L'aérosol
De Glaçage Émaillage
- Ⓔ DE Enamelite Sprühglasur Anweisungen
- Ⓔ IT Smalto spray Enamelite – Istruzioni per l'uso
- Ⓔ PT Instruções para vidrados enamelite aplicados
em spray
- Ⓔ ES Instrucciones del esmalte (barniz) en
aerosol Enamelite
- Ⓔ RU ИНСТРУКЦИЯ К ГЛЯНЦЕВОМУ СПРЕЮ ENAMELITE

Precaution: Allergic reactions may occur among the range of patients. Clinicians should screen patients for known allergies to plastic.

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Rx Only

MD

 **Keystone Industries**
52 West King St.
Myerstown, PA 17067 USA
800.333.3131
keystoneindustries.com

EC REP

Keystone Europe BV
Batavenweg 7
5349 BC Oss Netherlands

CE 1639

 **Keystone Industries GmbH,**
Stockholzstr. 11, 78224 Singen, Germany
+49 77 31 91 21 01

keystoneindustries.com/symbolglossary

5100452 ENALAB-004 Rev 1 12/2021

ENAMELITE SPRAY GLAZE INSTRUCTIONS

Indications: Enamelite Ceramic Spray Glazes are intended to be used for aerosol applications of ceramic porcelain glazes for dental restorations (crowns and bridges).

- Shake well until mixing ball moves freely inside can.
- Prepare restoration(s) in usual manner for stain and glaze.
- Use glaze on low fusing PFM porcelain, pressed ceramic, Lithium Disilicate and Zirconia restoration(s).
- Hold spray can upright 2-4 inches away from restoration(s) over dried and unfired stains.
- Use short bursts while shaking can between sprays on buccal, lingual, mesial, distal and occlusal surfaces
- If renewed application is necessary, spray restoration in the same manner again.
- If multiple units are placed on ceramic pillow, spray units simultaneously while shaking can often.

Firing Schedule:

Pre-Dry: 3-5 minutes

High Temp: 1346°F(730°C) -1562°F(850°C)

Start Temperature: 572°F/300°C

Vacuum: None

Firing Rate: 72°F/40°C

Cool Time in Firing Schedule: None

CTE: $10.0 \pm .5 \times 10^{-6}/^{\circ}\text{C}$

Hold Time: 1 minute

NOTE: If glaze should settle on the underside of restoration(s), dust with small bristle brush. Clear valve after each application by turning can upside down and depressing spray button.

WARNING: Do not inhale aerosolized ceramic dust. Do not get dust or liquid in contact with eyes or skin. Personal Protective equipment such as gloves, goggles, mask and lab coats are recommended when using this product.

Type I, Class 1b

Refer to product SDS for additional health, safety and disposal information.

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