Bosworth Original Truliner[®]

PMMA Denture Corrective Relining Material

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Type: Autopolymerizable polymer

Class: Powder and liquid pour-type

Keystone Industries
52 West King St.
Myerstown, PA 17067 USA
800.333.3131
keystoneindustries.com

keystoneindustries.com/symbolglossary

Danger

Liquid // Highly flammable liquid and vapor. Causes skin irritation. May cause an allergic skin reaction. May damage the unborn child. Suspected of damaging fertility. Suspected of causing cancer. May cause respiratory irritation. May cause damage to organs through prolonged or repeated exposure. **Powder** // May form combustible dust concentrations in air. Causes serious eye irritation. May cause an allergic skin reaction. Suspected of damaging fertility or the unborn child. Refer to SDS for additional Health, Safety and Disposal Information.

Warning: This product can expose you to chemicals including, N,N-dimethyl-p-toluidine which is known to the State of California to cause cancer, and dibutyl phthalate, which is known to the State of California to cause birth defects or other reproductive harm. For more information go to www.P65Warnings.ca.gov.

NOTE: Original and New Truliner Powders and liquids cannot be interchanged.

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
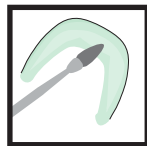
Bosworth Original Truliner Instructions for Use

Original Truliner is a PMMA denture corrective relining material used to restore a denture's original fit.


DIRECTIONS

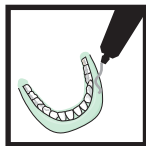1. Clean and dry denture. Roughen denture surface (2mm) and periphery for best adhesion.

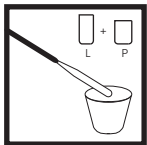2. Coat buccal and labial surfaces with Bosworth Masque.


3. Slowly add 1 full plastic vial measure of powder (15ml) to 1 full glass vial measure of liquid (7.6ml) in mixing cup. Spatulate for 30 seconds until material is a honey-like consistency.

4. Apply mixture to all surfaces to be bonded. When material is flat in color and has absorbed all liquid, rinse patient's mouth and insert denture.

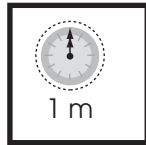5. Have patient close to align occlusion.

6. **Keep denture in patient's mouth until material is in a dough-like stage (approx. 1 minute).**

7. Remove from patient's mouth before final set. Rinse mouth with cold water. Adjust as needed and polish.

NOTES

1. Before making a permanent denture reline, check for sores or cancer and treat the tissue with a tissue conditioner such as Bosworth Softone to eliminate the possibility of the final reline duplicating the disturbed or deformed mucosa caused by the deficient denture.
2. Recommended: Bosworth's Pressure Pot for a smoother, color-stable and reduced porous reline.
3. **This product generates heat during polymerization that could be harmful to the patient or dental professional. Immediately remove from mouth when the patient senses the rising heat. Do not cure material to completion in mouth.**

See SDS for safe handling precautions and proper waste disposal.
(www.keystoneindustries.com/en/msds)

CAUTION: Certain individuals may be sensitive to products containing acrylic.

MIX RATIO

7.6ml (1 full glass vial) liquid to
15ml (1 full plastic vial) powder

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