Bosworth Glaze®

Self-Cure Acrylic Resin Polish

- FR** Glaçure de finissage pour appareils d'orthodontie, prothèses, couronnes, rebasages et restaurations temporaires
- ES** Esmalte para acabado para aparatos ortodóncicos, prótesis dentales, coronas, revestimientos y aplicaciones temporales
- DE** Finierlack Für die Orthodontie, Zahnprothesen, Kronen, Unterfütterungen und Provisorien
- IT** Smalto per finitura per apparecchi ortodontici, dentiere, capsule, ribasature e provvisori
- PT** Verniz de acabamentoo para aparelhos ortodónticos, dentaduras, coroas, revestimentos e provisórias

REF 0921236

Danger

Highly flammable liquid and vapor. Causes serious eye irritation. Causes skin irritation. May cause drowsiness or dizziness. May cause an allergic skin reaction (Glaze Liquid Only). Refer to product SDS for additional Health and Safety information.

 Keystone Industries
52 West King St.
Myerstown, PA 17067 USA
800.333.3131
keystoneindustries.com

keystoneindustries.com/symbolglossary

BOSLAB-308 PR124 Rev 1 09/2022

Bosworth Glaze Instructions For Use

1. Finish and polish restoration. Clean and dry surface. Dip brush into Glaze bottle.

2. Coat composite or acrylic surface with one or two straight, even strokes. Allow Glaze to dry 10-20 seconds before applying second coat, if desired.

3. Clean brush with Glaze Solvent after each use. If removal of Glaze is necessary, grind off with polishing wheel.

TO MAINTAIN A FREE-FLOWING CONSISTENCY

1. Glaze should be kept thin with periodic addition of Glaze Solvent.
2. Keep bottles sealed tightly.

TO USE GLAZE FOR AN IDENTIFICATION SYSTEM

1. Roughen area where name is to be placed.
2. Write patient's name on surface with a very fine permanent marking pen.
3. Cover roughened area with Bosworth Glaze.

See SDS for safe handling precautions and proper waste disposal.

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